## ASAND PROPHYLAXIS POWDER (EN)

Cosmetic product for mechanical cleaning of teeth with a non-abrasive cleaning system.  
Unique calcium microparticles for supragingival polishing with air.

### Features:

Cleans down to the enamel of the tooth and leaves the tooth whiter and smoother. Removes debris and stains on teeth.

### Use:

Open the bottle by turning the lid counterclockwise. Shake the bottle with powder before filling the applicator reservoir. Approximately 15 g is required per application. After dosing, close the bottle with a lid. When applying the powder to the teeth, follow the instructions of the applicator used.

### Additional use and safety information - warning:

For safety reasons, safety goggles must be worn by both users and customers. The product must not be used for people with respiratory diseases. Customers with contact lenses should wear additional eye protection. In extreme cases, especially with the presence of pathological periodontal sacs (> 6 mm), mucosal lesions and extremely long direct exposure to soft tissue, emphysema may occur. Root cement, demineralized enamel and fillers should not be directly blasted. Cleaning should only be performed here indirectly through the rolling effect. We recommend using mouth protection during work. All parts and products with sensitive surfaces should be removed from the treated area. The treatment can be supplemented by polishing the teeth.

### Notes for user:

The product is intended for professional use only for dental treatment in the field of dentistry.

Use for other purposes is not permitted and may lead to danger.

The powder should be used by a competent trained user only for the described application by the selected applicator, taking into account the applicable occupational safety regulations, applicable preventive measures and these instructions.

It is the responsibility of the user:

- only use fault-free consumable materials
- ensure use for the right purpose
- follow the method of application according to the instructions of the selected applicator
- protect yourself, the customer and third parties from danger

**Storage:** Store in a cool, dry place. Protect from heat (> 60 ° C).

**Ingredients:** Calcium Carbonate, Ammonium Nitrate, Aqua, Ammonium Phosphate, Dicalcium Phosphate, Hematite Powder

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